
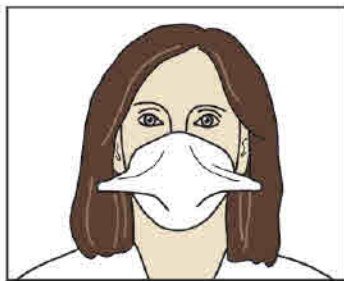
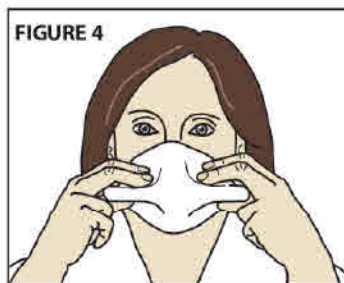
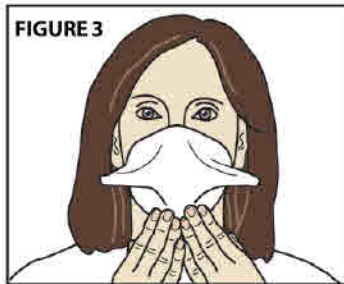
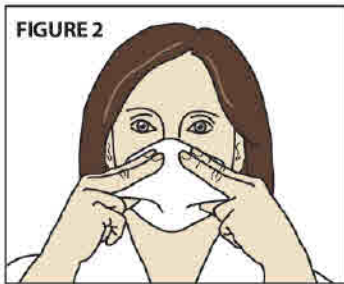


## USER INSTRUCTIONS<sup>†</sup>

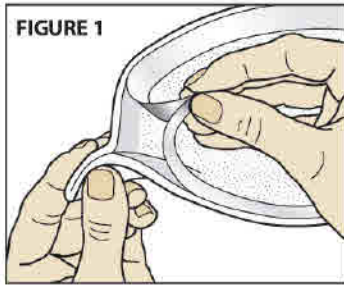

<sup>†</sup>Instructions must be followed each time the ViraMask™ is worn. Do not apply over full facial beards, cuts, abrasions or rashes.

### APPLYING THE VIRAMASK™

**Fig 1:** Holding the pull tabs, peel off ALL of the protective release liner from the back of the mask exposing the adhesive.

**Fig 2:** Position the top of the mask across the bridge of the nose using the alignment fold. Press firmly in place (for about 15 seconds).

**Fig 3:** Open mouth slightly to allow for talking and position the bottom of the mask to the bottom of the chin using the alignment fold. Press firmly in place (for about 15 seconds).

**Fig 4:** With top and bottom of the mask in place, press the pinched portions firmly against the facial skin to provide a leak-free seal in this area.

Continue pressing firmly around the perimeter of the mask, ensuring there are no gaps or wrinkles so that the mask forms a smooth continuous surface to the skin. Following these instructions creates a personalized custom face-seal fit to your facial structure while still allowing for ease of breathing and talking.

### REMOVING THE VIRAMASK™

**Step 1:** With two fingers, press the skin next to the pinched section of the mask to prepare for its removal.

**Step 2:** With the other hand, STARTING AT THE CHIN, PEEL UPWARD, “low and slow” keeping it close to the skin surface to avoid skin damage. As the mask is removed; continue moving fingers as necessary, supporting the newly exposed skin by pressing on it while removing the mask.

**HELPFUL HINTS:** Applying a barrier film to the face prior to applying the ViraMask™ can help protect against damage or irritation to the skin during its removal. Upon removal, wrinkle nose and facial muscles while slowly removing the mask to break the adhesion bond. Enhanced fit factors can be achieved by sealing the sides of the mask against the cheek areas with strips of sealant tape, preferably 3M™ Durapore™.

**USER SEAL CHECK:** According to OSHA manufacturer’s allowable preference, with the hands covering most of the mask, positive pressure will displace the mask slightly forward away from the face when the user breathes out forcefully. If leaks are detected, apply more pressure to the area of the leak to firmly set the adhesive.

**FIT TESTING:** In accordance with OSHA requirements, the user should have the ViraMask™ fit tested quantitatively with a Portacount Plus Fit Tester.